TALKJOY[®]

Dry-Smart - User Manual

DEUTSCH – ENGLISH – SPANISH – ITALIAN - FRENCH

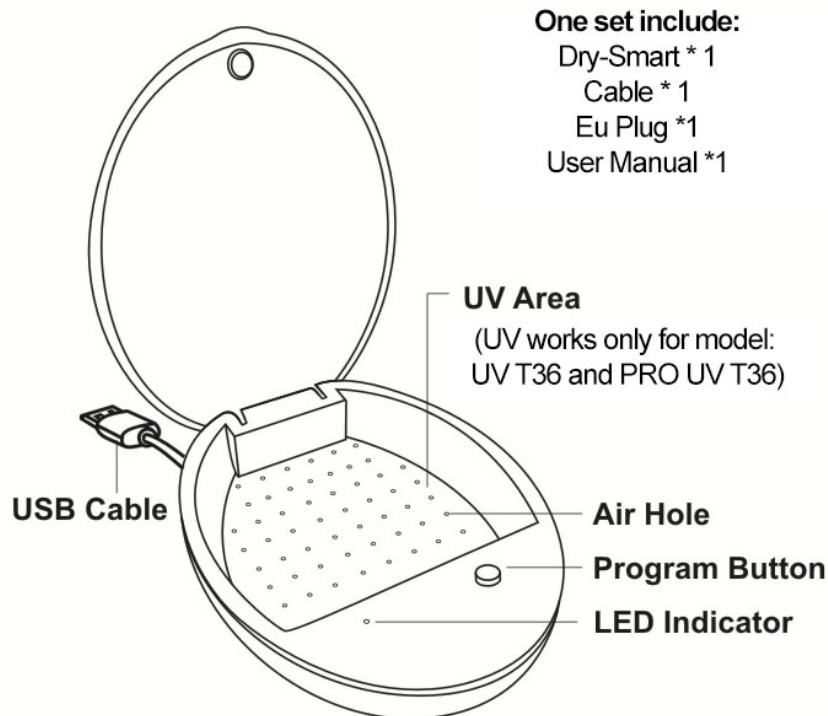
Dry-Smart, the electronic dehumidifier box is a good choice to care for your hearing aid and earmould.

The digital chip and temperature sensor in Dry-Smart will constantly measure and control the heating temperature.

Models with UV function will help killing germs, fungi and bacteria.

Using Dry-Smart correctly and regularly could extend the lifetime of your hearingaid.

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Attentions

1. Please clean the dirt and humidity from the hearing device, before you start drying.
2. To get a better drying effect, please get off the sound tube and open the battery door if your hearing device allows.
3. Please power off your hearing device before start work if it's rechargeable model with battery fixed in.
4. When drying finishes, please power off the Dry Smart.
5. Please do not disassemble the product by yourself.
6. UV starts working only when you shut the cover of the Dry-Smart and it stops working when you open the cover. This design is intended to protect your eyes. Long-term staring at the UV light may hurt your eyes.
(UV works only for model UV T36 and PRO UV T36.)

Usage Instruction

Get the Dry-Smart powered: connect Micro USB end of the cable with the Dry Smart, plug the cable into an adaptor and plug your adaptor into an electrical outlet.

Please read the following operating instruction according to the model you buy :

Model T3

- Get the Dry-Smart T3 powered, press the program button to switch on, the blinking blue indicates 3 hours' dry is on. You will see the stable blue light when drying finished.
- Press the program one more time, the Dry-Smart T3 is off.

Model T36

- Get the Dry-Smart T36 powered, press the program button to switch on, the blinking blue indicates 3 hours' dry is on. You will see stable blue light when drying finished.
- If you prefer to switch to 6-hours dry, please press the program button, the indicating light shows blinking green.
- When the blinking light switches to stable light, the drying process is finished. Please press the program to switch off and get your hearing device out.

Model UV T36 / PRO UV T36

- Get the Dry-Smart powered, press the program button to switch on, the blinking red and blue indicates 0.5 hour UV + 3-hours dry is on. When UV finishes, 3 hours dry starts automatically, you see blinking blue light. When 3-hours dry finished, the blue light will turn to stable.
- If you prefer to have your hearing device enjoy a 6-hours dry, please press the program button one more time, the blinking red and blinking green indicates 0.5 hour UV + 6 hours dry. The green will become stable when drying finished.
- Press the program third time, the Dry-Smart is off.

Below table may help you understand the light indication:

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